



INSTRUCTIONS FOR USE


MOZAÏK / PAD3434-2




1. Remove the tape completely from the fistula needle.




2. Partially remove the needle from the cannulation site without exerting any compression on the site.

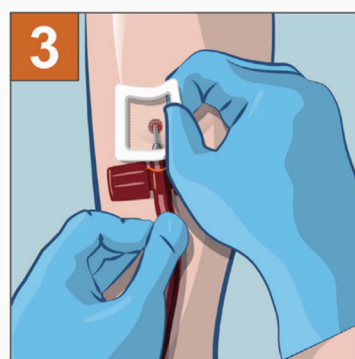

3. Apply Mozaik to the accessible area of cannulation site, ensuring adhesion of the foam frame to the patient's skin and ensuring there are no skin creases under Mozaik.

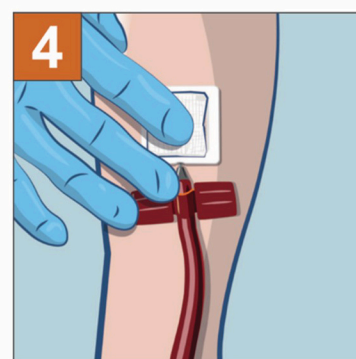


4. Place sterile gauze gently on top of Mozaik.

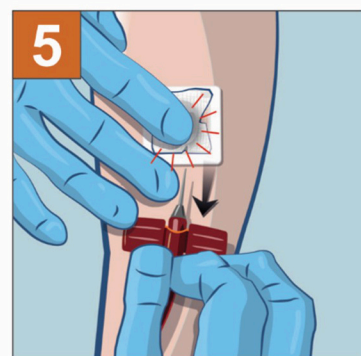

5. Remove the needle entirely, compressing immediately with 2 fingers after needle removal to avoid damage but optimise haemostasis



6. Pressure should be mild and direct over the site, compression should be maintained for 3 minutes

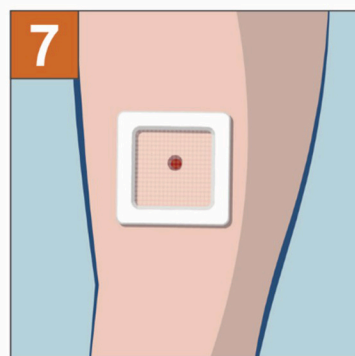

7. After 3 minutes gently lift the gauze to inspect the haemostasis status through the transparent window of Mozaik, if oozing continues, compress again and inspect every minute thereafter until haemostasis is achieved.



8. Mozaik can be removed from the cannulation site between 2 and 24 hours after haemostasis was achieved.



REF 07920594100 / V3 / 2023-04-14



Mozaik™ in reality


Discover now the Mozaik™ device in video

*According to EVAM0Z study, ECHO, 2024

COAGULATION



Microperforated film dressing for significantly reduced time to haemostasis

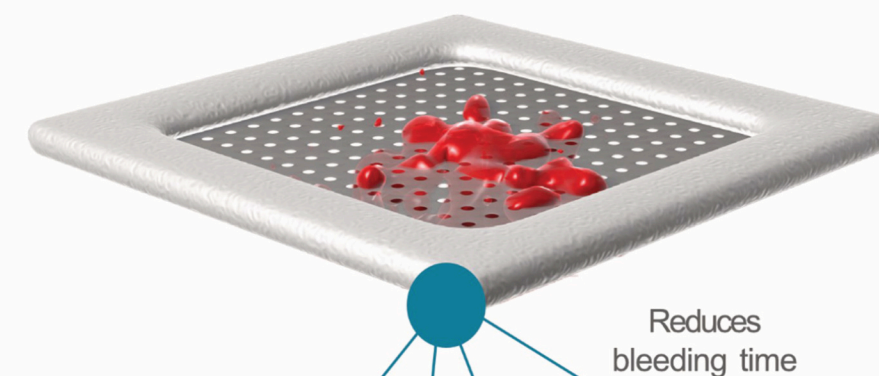

Reduces bleeding time

Enables real-time monitoring of the coagulation process

Thick foam frame for easy installation and prevention of blood leakage

Free of chemical agents and haemostatic substances



-54%

Reduces compression time by 54% vs standard protocols*

MADE IN FRANCE



NEPHROKIT, chez Gergonne Industrie
ZI nord - Rue de Tamas - CS 70204 01117 Oyonnax Cedex - France
www.nephrokit.fr