


# SolüKAT™

All-in-one medical adhesive device for Central Venous Catheter (CVC) protection.

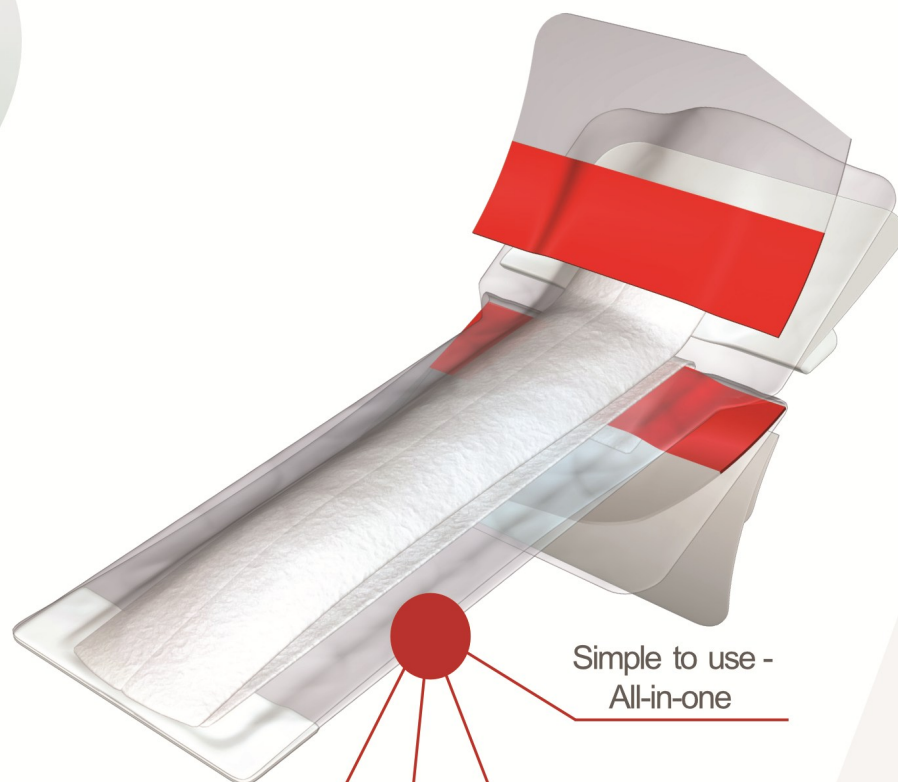

Simple to use -  
All-in-one

Reduces the risk of  
exit-site infection

Very comfortable -  
soft and discreet

Waterproof, allows  
showering



Solukat™ in reality

Discover now the Solukat™  
device in video

Solukat™ is an easy-to-use, single-use, sterile device designed to fully protect a central venous catheter in compliance with the strictest hygiene protocols.


Solukat™ is made of polyurethane, a material with a soft touch and unique characteristics since it is simultaneously a bacterial barrier, waterproof and highly permeable to gases (breathable).

Applied at the end of the dialysis session, Solukat™ can remain on the skin for up to 3 days, providing adequate protection from the exit sites to the connectors until the next session.

Improving patients' quality of life is our priority, that is why Solukat™ is a one-piece device perfectly sealed that allows the patient to enjoy showering again. To limit the risk of irritation, Solukat™ adhesive is biocompatible, and we also reduced the surface in contact with the skin to a minimum.

Patients with the most sensitive skin should use Solukat Si™, provided their skin does not perspire profusely, naturally, or as a result of sporting activities.

## INSTRUCTIONS FOR USE

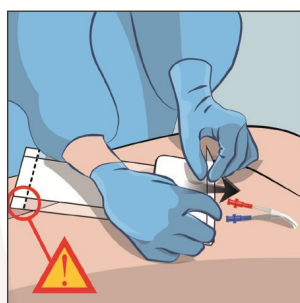


Slide the catheter ends into the bag and up to the insertion points.

⚠ Place the adhesive strip on the skin side.




Remove the protective paper and stick the upper part.

Remove the protective paper from the lower part.

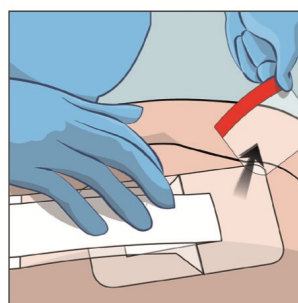

Remove the red marking strips.

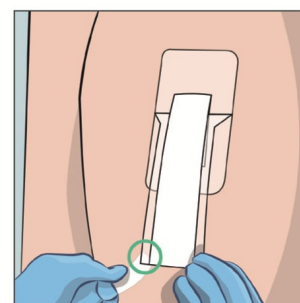

Remove the protective paper from the adhesive strip. Solukat™ is now ready for use.

Solukat is available in boxes of either 100 units or 12 units.

The product codes are as follows:

PADS 1000-100: 100 units per box

PADS 1000-12: 12 units per box (minimum order is two boxes)

[orders@xtra-med.com](mailto:orders@xtra-med.com)

Distributed by: XtraMed

[www.xtra-med.com](http://www.xtra-med.com) [info@xtra-med.com](mailto:info@xtra-med.com)

Deborah Brady 07717 660461

Steve Lane 07885 597850

