


SölüKATTM



Medical device with silicone adhesive for Central Venous Catheter (CVC) protection and allowing exit site monitoring.

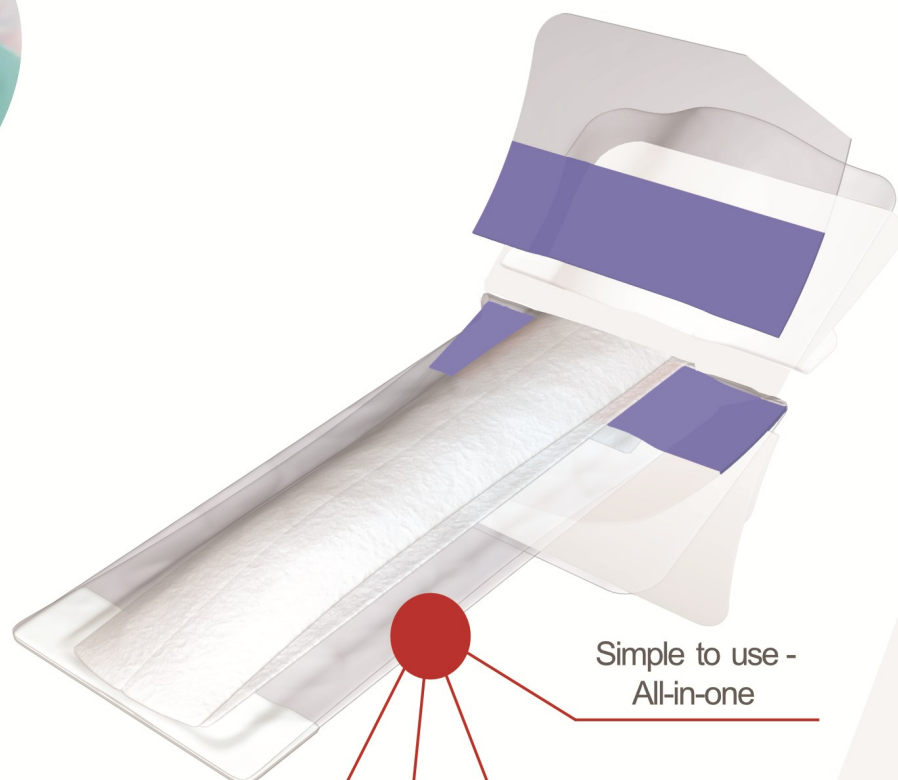

Simple to use -
All-in-one


Allows exit site monitoring

Waterproof, allows
showering

Silicone adhesive - Very comfortable
and atraumatic removal



MADE IN
FRANCE

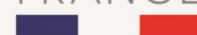

nephrokit
innovation

Solukat Si™ is an easy-to-use, single-use, sterile device designed to fully protect a central venous catheter in compliance with the strictest hygiene protocols.


Solukat Si™ is made of polyurethane, a material with a soft touch and unique characteristics since it is simultaneously a bacterial barrier, waterproof and highly permeable to gases (breathable). Its asymmetrical pads allow to choose to leave the exit site visible for better monitoring or to hide it for more discretion.

Applied at the end of the dialysis session, Solukat Si™ can remain on the skin for up to 3 days, providing adequate protection from the exit sites to the connectors until the next session. The silicone adhesive of Solukat Si™ will preserve the most sensitive skins by increasing user comfort and guaranteeing painless removal.

Improving patients' quality of life is our priority, that is why Solukat Si™ is a one-piece device perfectly sealed that allows the patient to enjoy showering again.

Solukat Si™ is not suitable for skin that perspires profusely, naturally, or as a result of sporting activities. We recommend using Solukat™ when this is your case.


INSTRUCTIONS FOR USE

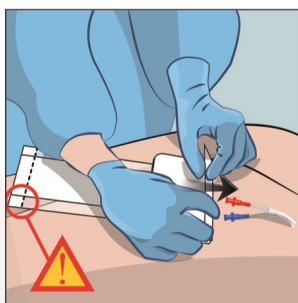

Slide the catheter ends into the bag and up to the insertion points.




Place the adhesive strip on the skin side if you want to see the exit site. Otherwise, place the adhesive strip on the other side.

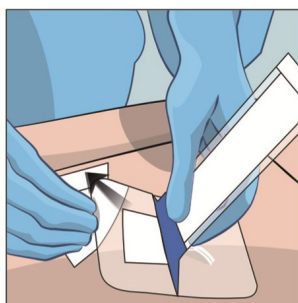


Remove the protective paper and stick the upper part.

Remove the protective paper from the lower part.

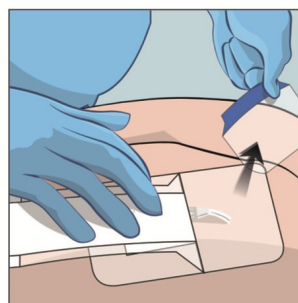

Remove the blue marking strips.

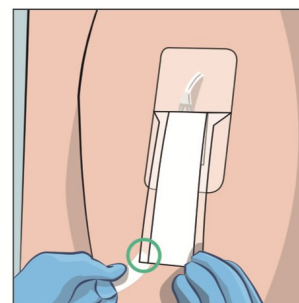

Remove the protective paper from the adhesive strip. Solukat Si™ is now ready for use.

Solukat Si is available in boxes of either 100 units or 12 units.

The product codes are as follows:

PADS 1000SI-100: 100 units per box

PADS 1000SI-12: 12 units per box (minimum order is two boxes)

orders@xtra-med.com

Distributed by: XtraMed

www.xtra-med.com info@xtra-med.com



Deborah Brady 07717 660461

Steve Lane 07885 597850